



STEP BY STEP

CLEAN&SEAL® offers a two-phase subgingival treatment, both in the therapy of periodontal and peri-implant infected pockets.

CLEAN: PHASE 1 – WITH PERISOLV®



PERISOLV® must be cooled for storage.



Do not rinse with CHX-containing products before, during and after treatment. (CHX = chlorhexidine)

1


MIXING OF THE SYRINGES



Use the information provided in the package insert and the visualization on the product box for the preparation of the cleaning gel with the components in the two syringes.

2

INJECT

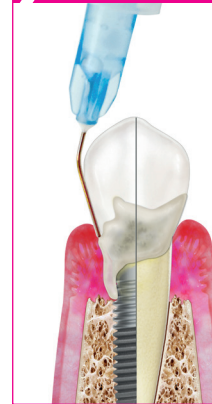

- Please fill the pockets to be treated with the cleaning gel.
- Avoid overfilling the pockets.

3

PERISOLV® ACTS

Leave the cleaning gel > 60 sec in the pocket.



4

MECHANICAL DEBRIDEMENT



Thorough instrumentation of the pocket and the tooth root surface. Remove the detached granulation tissue with tweezers.

5

RINSING



After mechanical cleaning, rinse the pocket with saline solution (NaCl) until no visible debris or biofilm residues are flushed out of the pocket.



6

ADDITIONAL CLEANING CYCLE

- Fill the pocket again with PERISOLV® as long as debris or biofilm residues are flushed-out of the pocket during step 5.
- Perform another instrumentation.
- For periodontal pockets, applying 3-4 cycles is common. For peri-implant pockets, doing 4-7 cycles is usual.



7

RINSING



Please rinse the pocket thoroughly with saline solution (NaCl) to remove possible PERISOLV® residues.





STEP BY STEP


SEAL: PHASE 2 – WITH hyaDENT BG

1

PREPARATION



- > The preparation of hyaDENT BG is carried out according to the enclosed package insert.
- > Use rounded needle. E.g.: 27G 0.4 mm x 20 mm

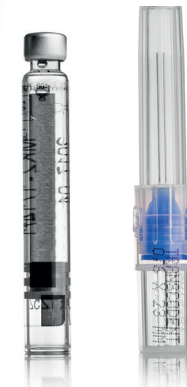

2

SEAL THE POCKET



- > Fill the pocket from apical to coronal with hyaDENT BG



After the treatment, the patient should **not eat or drink anything for two hours**, so that hyaDENT BG can start the healing process.



Do not use **mouth rinses** with low concentration chlorhexidine until 2 days after treatment.

During the first week after the **CLEAN&SEAL®** treatment, the patient should refrain from **tooth- and interproximal brushes**.



3

RECALL

- > If possible, perform a refill of the pocket with hyaDENT BG at the follow-up after a few days.



4

RE-EVALUATION

Involve the patient in regular checks. For example plan a follow up with the patient at **3 and 6 weeks** for the healing check.

Please follow the EFP recommendation and refrain from measuring the pocket depth during the first 3 month.



PATIENT RECALL

After approximately **12 weeks**, the clinical situation should be assessed to check how the clinical parameters (BOP, CAL, PD) have evolved.